# MANUAL BOOK

---

OMBMAK0068

Eff. Date: 08/04/2021

## DAFTAR ISI/CONTENTS

---

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*Tray is made of stainless steel sheet with list on each side.*

*Strong and anti-corrosion stainless steel section.*

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*Height position can be adjusted by using pin lock system.*

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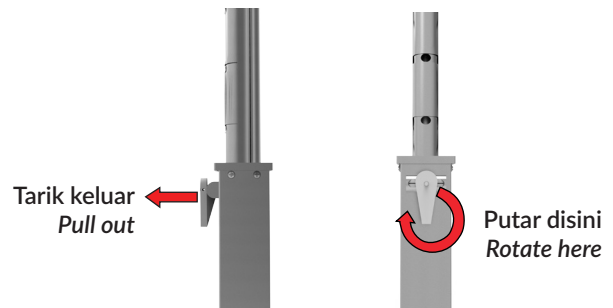
### Pembersihan Rutin / Regularly Cleaning

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*Don't place any item above a maximum load (15 kg total) on the mayo stand.*

Jangan meletakkan benda-benda selain peralatan medis pada meja mayo.

*Don't place any materials except medical apparatus.*

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*Remove the product when moving up the bed to avoid the patient to be bind.*

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*List and chemicals concentrations could effect on the surface of powder coating and stainless steel.*

No	Chemicals	Concentration
1	Acetyl Chloride	100 %
2	Acetic Acid	>10 %
3	Aluminum Chloride	>10 %
4	Bromic Acid	Concentrated
5	Chloric Acid	100 %
6	Chromic Acid	>2 %
7	Lactic Acid	>10 %
8	Lime Chloride	Diluted
9	Hydrogen Chloride	>1 % - Unhydrous
10	Hydrochloric Acid	15 %
11	Sulfuric Acid	15 %
12	Nitric Acid	10 %
13	Phenol	5 %
14	Sodium Phosphate	10 %
15	Pickling Solution	
16	Water, Sea	

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*Keep away from chemicals (acid/alkali) that could damage the materials*

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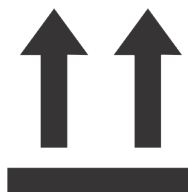
Peringatan & Perhatian  
*Caution and Attention*

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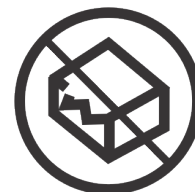
*These Manual Service Book is used as periodically maintaining guidance, and/or if any function defect/detriment of product.*

### **Langkah-Langkah Pelaksanaan Servis Berkala/Ways Of Periodically Maintaining Guidance :**

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*Prior to doing routine inspection, make sure the technician already known its product specification – see Manual Book.*

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*Make sure inspection done while product not in use and no added items as well.*

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*Aspect indicated component inspection related to any detriment.*

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*Numerical in "Qty" indicated component quantity related to product pertinent.*

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*Picture and arrow head used to ease the technician in order to see component's position that will be checked.*

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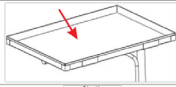
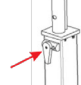
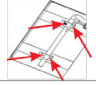
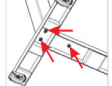
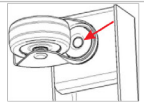
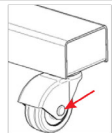
*Period indicated period time for routine inspection toward every components.*

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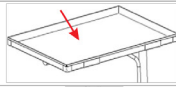
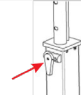
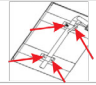
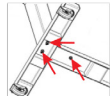
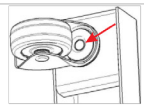
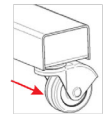
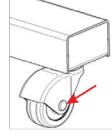
*Action indicated component reparation processing if any aspect insatiable.*

No	Inspection Items	Acceptance Standard	Qty	Picture	Period	Action / Recommendation
1	Overall components/items	Cleanness, corrosion, dried, overall function, smoothness	-		1 week	Clean table top and overall, dry, check smoothness
2	Table top	Cleanness, dented	1		1 year	Clean after use/daily, repair
3	Hi-lo adjuster lock	Tightening, worn out, broken, sluggish	1		6 months	Fasten, change if worn out/broken, grease if needed
4	Crossbar bolt to table top (Hex.bolt M8x20mm)	Tightening, worn out/broken	4		6 months	Fasten, change if worn out/broken
5	Pedestal bolt to stand (Inbush bolt M6x15mm)	Tightening, worn out/ broken	3		6 months	Fasten, change if worn out/broken
6	Castor bolt to pedestal	Tightening, worn out, broken	4		3 months	Fasten, change if worn out/broken
7	Castor's wheel	Cleanness, broken	4		3 months	Clean, change if broken
8	2" castor & castor bolt	Tightening, worn out, sound when moving, can't be rotated, loose, broken, sluggish	4		3 months	Fasten, change if worn out/broken, grease

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No	Inspection Items	Acceptance Standard	Qty	Picture	Period	Action / Recommendation
1	Overall components/items	Cleanness, corrosion, dried, overall function, smoothness	-		1 week	Clean table top and overall, dry, check smoothness
2	Table top	Cleanness, dented	1		1 year	Clean after use/daily, repair
3	Hi-lo adjuster lock	Tightening, worn out, broken, sluggish	1		6 months	Fasten, change if worn out/broken, grease if needed
4	Crossbar bolt to table top (Hex.bolt M8x20mm)	Tightening, worn out/broken	4		6 months	Fasten, change if worn out/broken
5	Pedestal bolt to stand (Inbush bolt M6x15mm)	Tightening, worn out/ broken	3		6 months	Fasten, change if worn out/broken
6	Castor bolt to pedestal	Tightening, worn out, broken	4		3 months	Fasten, change if worn out/broken
7	Castor's wheel	Cleanness, broken	4		3 months	Clean, change if broken
8	2" castor & castor bolt	Tightening, worn out, sound when moving, can't be rotated, loose, broken, sluggish	4		3 months	Fasten, change if worn out/broken, grease

If there are any problems, please see Manual Assembly, Part Catalog etc.

Read the manual book carefully first before using or operating this product